



## Responsive Respiratory, Inc.

[www.respondo2.com](http://www.respondo2.com)

**TO REORDER 866-333-4030**

Part: 180-5004/ 180-5005/ 180-5006

### LITER METER INSTRUCTIONS

- (1) Connect to oxygen source using standard oxygen tubing with end adapters.
- (2) Connect other end of oxygen hose to the inlet of the Liter Meter located as the bottom (away from pen clip).
- (3) Set oxygen source to a specific reading that falls within the Liter Meter scale range.
- (4) Hold the Liter Meter in a vertical position and take reading. Liter Meter must be held as vertical as possible to insure a correct reading. When reading the Liter Meter, use the center of the ball for actual reading against indicator lines.

#### ***Accuracies and Specifications***

180-5004: 1/8-2 1/2 LPM, +/- 0.125 LPM

180-5005: 1-8 LPM, +/- 0.25 LPM

180-5006: 2-15 LPM, +/-0.4 LPM

**Note:** Liter Meters are used to test approximate accuracy of your oxygen equipment. Liter Meters should be checked for accuracy every 6 months of usage against a known standard.